



RECOMMENDED USE AND CARE OF BLUE DIAMOND® GEL PADS

CLEANING: Use of standard operating room cleaning agents. Long soaks not recommended.

SHARPS: Avoid using sharp instruments that may puncture the outer film. Do not drag pads and positioning devices across sharp edges.

WARNING: BLUE DIAMOND® gel pads may be warmed in warming cabinets or by immersing in water to make the pads approach normal human body temperature.

DO NOT AUTOCLAVE or ATTEMPT TO STERILIZE the products.

Gel pads should not be exposed to high heat or flame.

STORAGE: Store pads flat whenever possible. Do NOT store in cold places.

MOVING PATIENTS: It is not recommended to move patients while on the pads. Moving patients while on pads may result in tearing or puncturing of pads.

PROTOCOL: A suggested protocol for BLUE DIAMOND® pads are available from the factory.
A Health Care Professional should evaluate the patient pre-operatively to determine the use of gel pads or positioners.

WARRANTY: All Blue Diamond® Polymer products are covered by a two year manufacturers defective replacement policy. If pad is deemed defective due to manufacturing process it will be replaced at no cost.

QUESTIONS: Contact the distributor from whom the product was purchased.