

Vinyl Sand Bag

CLEANING & CARE INSTRUCTIONS

Thank you for choosing our pad for your radiology accessory needs. To ensure the longevity of your new sand bags, we recommend the following guidelines for cleaning and care.

Cleaning & Care:

Do:

- It is recommended to clean with cold water and a mild detergent immediately after contact with fluids. Household 409 Spray can be used to clean the surface fabric or vinyl and remove dust and dirt from your sand bag.
- To disinfect your sand bag, it is recommended to use disinfectant wipes safe for use on vinyl materials. Allow a drying time of at least 3 minutes.
- Store your sand bag on a rack when not in use.

Don't:

- Do not use Bleach or any harsh chemicals on your sand bag.
- Do not autoclave or launder.
- Do not soak/submerge sand bag in water.

Failure to adhere to the above recommendations will shorten the life of your sand bag, promote cracking, and will possibly void your warranty.